

Venus® Bulk Fill

Simple, fast, safe!



NEW: One-layer simplicity
– ideal for milk teeth!

Venus®

Milk Teeth



One material
– One layer

Adult Teeth



Two materials
– Two layers

The flowable composite for greater efficiency in posterior restorations and milk teeth: Venus® Bulk Fill.

- **Efficiency:** Venus Bulk Fill permits easy filling of class I and II cavities in increments of up to 4 mm and is eliminating the need for complex, 2-mm layering techniques. Milk teeth even allow just one single layer.
- **Easy to use:** Self-adaptive handling properties provide ideal coverage of the cavity floor and walls. Simple and precise workability is the result.
- **Safety:** Long-lasting restorations due to outstanding curing depths and minimal shrinkage stress. Exceptional radiopacity guarantees a reliable diagnosis.

Giving a hand to oral health.

Venus® Bulk Fill – Simple, fast – safe!

For greater efficiency in posterior restorations

In everyday practice, when time is tight, a steady hand is still required. We continuously improve and expand our product portfolio with the aim of saving you time and optimising your treatment results. This is precisely the idea behind Venus Bulk Fill: a flowable, low-shrinkage composite for creating posterior restorations with a bulk-filling technique that is simple, fast and safe, and saves a lot of time.

One-layer simplicity ideal for milk teeth

In addition, Venus Bulk Fill is even easier and faster to apply on milk teeth. You do not need a second layer of any universal composite, which saves time and makes it ideal for the treatment of children. As such it represents a huge chance to give children a relaxed take-off into dental experience.

▪ Efficiency:

Venus Bulk Fill enables simple filling techniques in one layer of up to 4 mm for class I and II cavities. And for children, the treatment process ends here after the light curing. On adult teeth a final 2 mm layer of universal composite, such as Venus Diamond or Venus Pearl, is applied subsequently. Complex, time-consuming techniques using 2 mm layers are not necessary. This helps to save time and costs on each restoration, unlike traditional, laborious layering procedures. A total of 82% of surveyed dentists claimed that they saved up to 10 hours of chair time per month by using a bulk fill material.*

▪ Easy to use:

Perfect flow properties ensure easy coverage of the cavity floor and adaptation to the cavity walls. Venus Bulk Fill is compatible with (meth)acrylate-based adhesives and composites – no change in your trusted composite or adhesive is necessary. Venus Bulk Fill offers simple dispensing and placement via a convenient Pre-Loaded Tip (PLT) or a non-dripping syringe.

▪ Safety:

Venus Bulk Fill delivers a high depth of cure and safety for polymerisation of layers up to 4 mm thick. Low shrinkage means long-lasting restorations. High radiopacity guarantees a reliable diagnosis.

* Integrated media survey May 2010 in USA



Milk Teeth



Adult Teeth



To support you in giving children a relaxed and stress-free access to dentistry, we added a set of six finger puppets to the Venus Bulk Fill Pediatric Kits. These are designed to entertain the small patients during the treatment and might also help in explaining the treatment routine.

Order No.	Product	Content
66057342	Venus® Bulk Fill Syringe Pediatric Kit + iBOND® Self Etch	4 x 1.8 g syringes Venus® Bulk Fill – universal shade 1 x 2 ml bottle of iBOND® Self Etch 1 out of 6 finger puppets
66044743	Venus® Bulk Fill PLT Refill	10 x 0.2 g PLTs Venus® Bulk Fill – universal shade
66046795	Venus® Bulk Fill Syringe Refill	1 x 1.8 g syringe Venus® Bulk Fill – universal shade