SOPIRA® Carpule® Syringe Aspirating

DE	Gebrauchsanweisung.....	2
GB	Instructions for use.....	4
FR	Mode d'emploi.....	6
ES	Instrucciones de uso.....	8
IT	Istruzioni per l'uso.....	10
PT	Instruções de uso.....	12
NL	Gebruiksaanwijzing.....	14
SE	Bruksanvisning.....	16
DK	Brugervejledning.....	18
NO	Bruksanvisning.....	20
FI	Käyttöohjeet.....	22
GR	Οδηγίες Χρήσης.....	24
LV	Lietošanas instrukcija.....	26
LT	Naudojimo instrukcija.....	28

KULZER

mitsui chemicals group

Manufacturer:
Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)

Made in Germany

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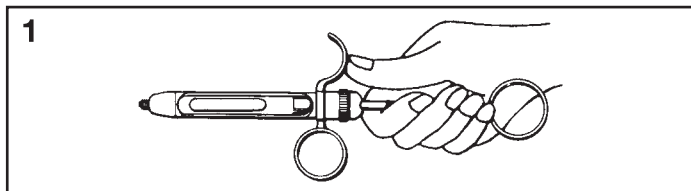
Aspirating dental cartridge syringe

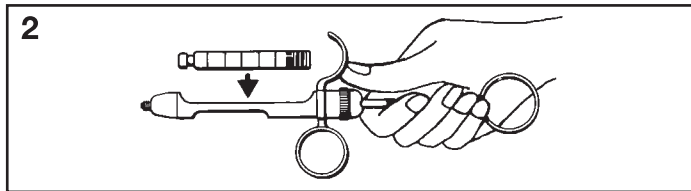
In combination with SOPIRA Carpule disposable needles the SOPIRA Carpule syringe provides a universal, easy-to-use injection system.

- All standardised cartridges (1.8 ml) with rubber plunger (solid as well as recessed) can be used with the SOPIRA Carpule Syringe.
- The rotatable cartridge holder permits perfect view of the cartridge in every situation, especially when controlling the aspiration process and the volume of the injected solution.
- The chromium-plated SOPIRA Carpule Syringe is easily to dismantle, allowing easy cleaning and sterilization.

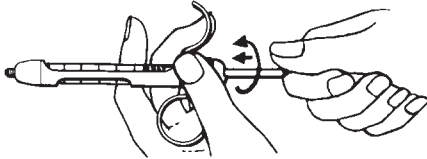
Instructions for use:

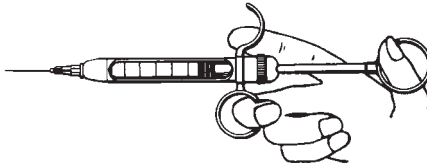
Withdraw the piston rod completely.

Load the cartridge.

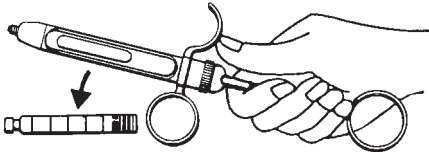
3

Screw the spiral hooks clockwise into the rubber plunger applying light pressure.

4

Fix the needle:

Ensure that the cartridge end of the needle pierces the centre of the rubber membrane. It is advisable to insert the needle just prior to injecting. Expel a few drops of the solution. The syringe is now ready for use.

5

After injection:

Detach the needle and remove the spiral hooks from the rubber plunger. Fully retract the piston rod and remove the cartridge.

Sterilisation:

After use, the SOPIRA Carpule Syringe can be sterilised in your usual equipment. Before sterilization the syringe should be cleaned with appropriate detergents.

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