




# standard staining of central incisor in 1 firing

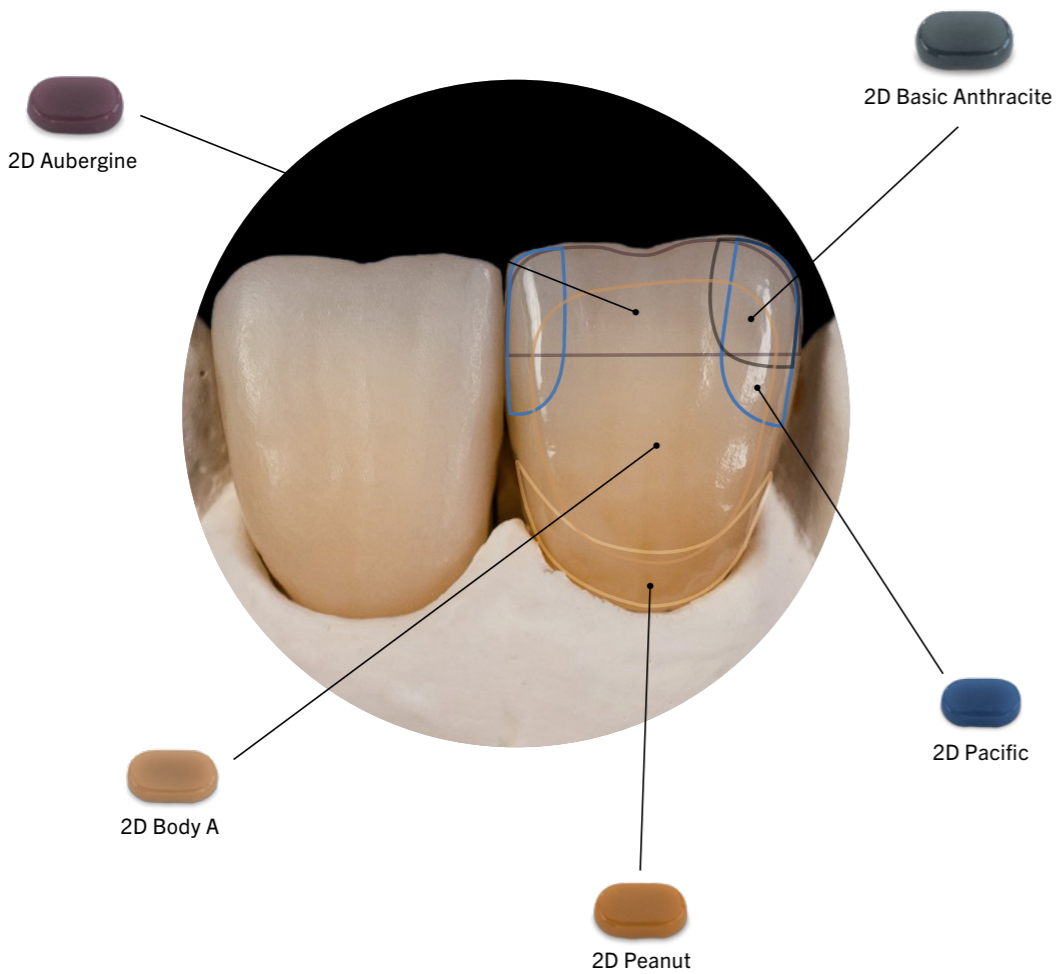


## Workflow:



2D Staining  
White Aesthetics



1. Initial situation: Monolithic multi-layer zirconium dioxide framework in A3 – ready for individualization

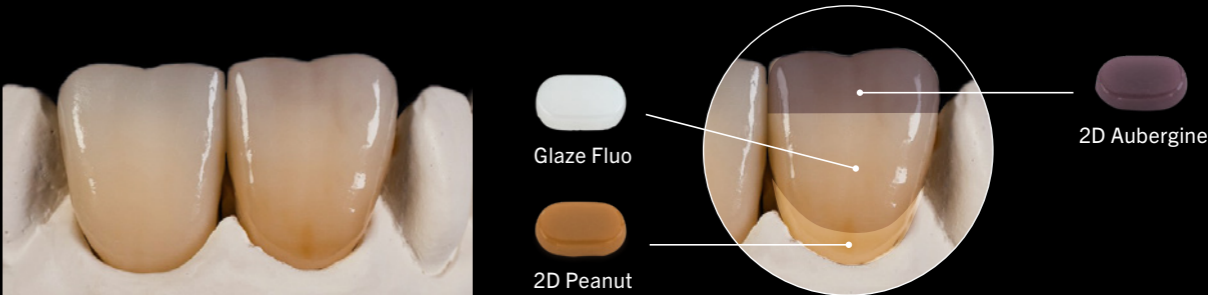

2. Thinly coat the entire surface with HeraCeram cre-active **Glaze Fluo** to gain fluorescence effect. Apply HeraCeram cre-active **2D Peanut** in cervical area to enhance the chroma. Create translucency effect in incisal area with HeraCeram cre-active **2D Aubergine**.



3. To enhance translucency effect: Apply HeraCeram cre-active **2D Basic Anthracite** distally and blend out towards the incisal edge. Apply HeraCeram cre-active **2D Pacific** mesially & distally and spread towards the incisal edge.



4. Staining of the body area with HeraCeram cre-active **2D Body A** to adapt tooth colour according to Vita® Classic Shade Guide.



Tooth 21: Initial situation/Tooth 11: Final result after staining with HeraCeram cre-active realised in 1 firing. Created by Dental Technician Thomas Backscheider.

\*VITA® is a registered trademark of VITA Zahnfabrik H.Rauter GmbH & Co. KG, Bad Säckingen, Germany



video  
tutorial